



## Individual Preset Magnetic-only and Massage-only Programmes

Prog	MAS PMT	Time t	Frequency F	Magnetic Intensity I	Laser L	Massage Duration d	Massage Intensity h	Comments
P01	PMT	10 min	12 Hz	10	12	ON	5	Higher/mid stim
P02	PMT	10 min	8 Hz	10	OFF	ON	6	Higher/mid stim
P03	PMT	10 min	8 Hz	10	10	ON	5	Mid stim
P04	PMT	10 min	6 Hz	10	OFF	ON	5	Mid stim
P05	PMT	10 min	6 Hz	10	6	ON	5	Mid/low stim
P06	PMT	10 min	4Hz	10	OFF	ON	5	Mid/low stim
P 7-18								Laser Cluster Programmes – See Sheet

Please watch for reactions and put on to a lower frequency setting if required.

### Recommended Pre settings for Pulsed Electromagnetic therapy (PMT) and Massage (MAS)

Note: preset treatment times of programs (P01 – P10) are adjusted to suit running the combination of programmes using sequences S01 – S08. To increase individual programme treatment times either run a sequence repeating the same program several times (e.g. P07, P07, P07 would be a 20 min massage treatment) or adjust one of the preset programmes. Programmes 11-18 can also be set and used on their own.

Contra Indications: pulsed electromagnetic therapy should not be used if the horse has: Tumour development, cancer, epilepsy, severe fungal diseases or is pregnant.

### Important notice

All referenced values are recommendations based on practical use. You should, however, closely observe your individual horse's reactions to treatments which may differ from the 'normal' reactions of our referenced values. Your vet or equine therapist may be able to assist in working out suitable treatments for your horse(s).